

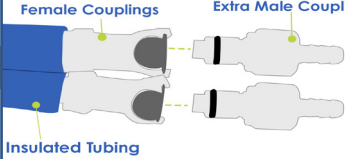
POLAR

P R O D U C T S



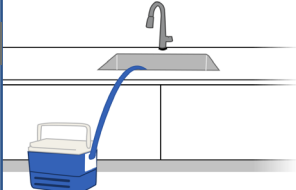
**Polar Products Active Ice[®]
Drain, Maintenance &
Storage Kit**

Empty The Cooling Reservoir Without Having To Lift It:

- 

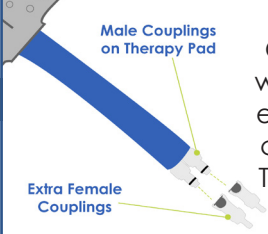
Female Couplings **Extra Male Couplings**

Insulated Tubing

Connect the extra male couplings into the female couplings that are attached to the insulated tubing coming from the cooling reservoir.
- 

Point the end of the insulated tubing into a sink, turn the unit on and it will pump out most of the water.

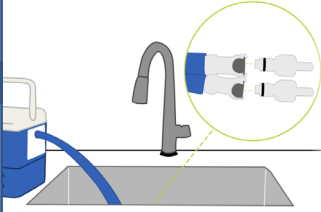
Preparing The System For Storage:

- 

Male Couplings on Therapy Pad

Extra Female Couplings

Drain The Therapy Pad

Over an area where draining water will not cause problems, connect the extra female couplings into the male couplings attached to therapy pad. This will allow water to drain from the therapy pad.
- 

Drain the Cooling Reservoir & Tubing

Empty the cooling reservoir of water. Elevate the reservoir and place the end of the tubing in a sink. Connect the extra male couplings into the female couplings at the end of the insulated tubing. This will allow excess water to drain from the reservoir and tubing.

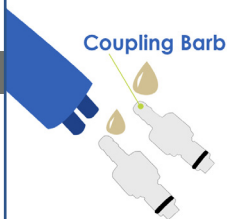
Replacing Couplings:

1



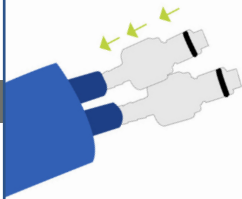
Cut off the old couplings with a pair of scissors. Always be sure you cut the tubing back to unused tubing. This will ensure a good seal. Trim back the blue foam insulation if necessary.

2



Put a small dab of glue on the coupling barb before inserting it into the tubing. Most any glue will work: PVC or Rubber Cement, Gorilla Glue, Super Glue, etc.

3

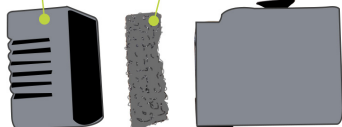


Push the new couplings into the tubing.

Replacing The Pump Filter

Pump Cover

Filter



Remove the pump cover to replace the filter

Components



(2) Female
Couplings



(2) Male
Couplings



(2) Pump
Filters

(6) O-rings



LET'S TALK

3380 Cavalier Trail
Stow, Ohio 44224
1.800.763.8423
polar@polarproducts.com



Have any questions?
Need help
troubleshooting? Proudly
designed, engineered,
tested and assembled in
the USA. Our customer
service is located at our
factory in Stow, Ohio
and a real person will be
available to talk from 8-5
EST.

Give Us A Call!